


# Canine: 5-Lead Holter Hook-Up Instructions

## Right Side

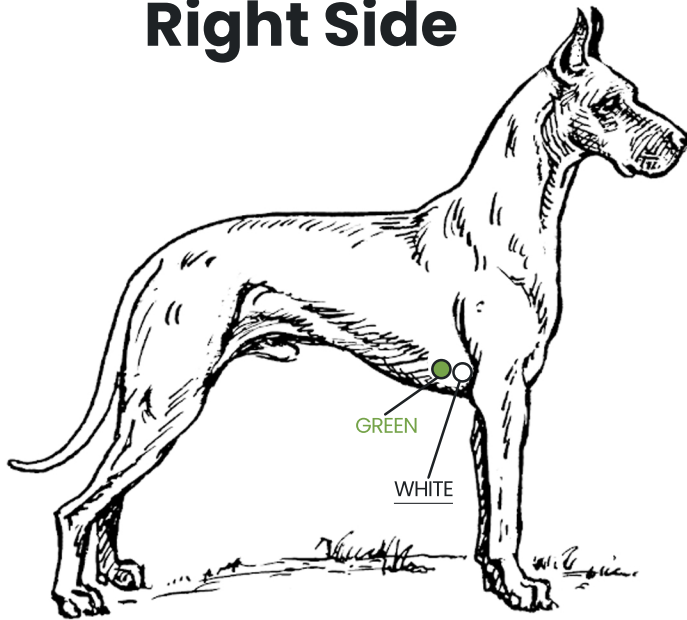

### WHITE: (RA)

Place electrode in front of the 5th rib over the pulse point behind and under the right front leg

### GREEN: (RL)

Place electrode between the 5th and 6th rib directly behind the WHITE electrode

### BROWN: (V)

Place electrode between the 5th and 6th ribs

### BLACK: (LA)

Place electrode above the BROWN electrode

### RED: (LL)

Place electrode between the 6th and 7th ribs, directly behind the BROWN electrode

## Left Side

