


DO NOT PUT BATTERY IN THE MONITOR UNTIL YOU ARE READY TO ACTIVATE THE MONITOR

H3+™ DIGITAL HOLTER RECORDER

To Begin a Session

1. Attach the patient cable to the H3+.
2. Remove the battery door of the H3+, insert a new AAA battery, and close the battery door.

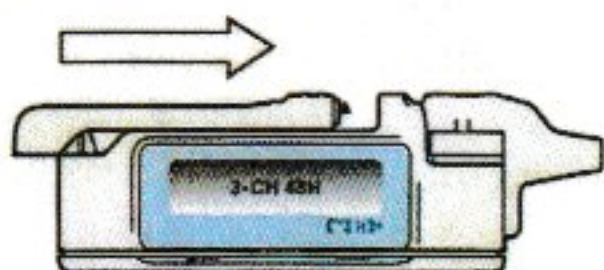


3. Press the **Enter** button to move through the menu displays.



- Verify the current time setting.



- Verify the ID number.

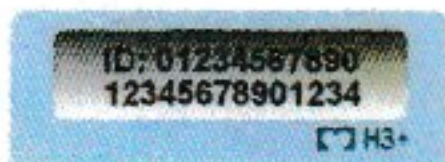


- Verify the ECG signal quality.




Note: Waveform is shown at 4 mm/mV gain to allow full representation of ECG.

4. To begin recording, press and hold the **Enter** button for a period of 3 seconds. An **R** indicates Recording Mode.

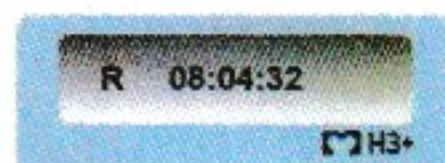

Entering Optional Diary Events

To mark an event, press the **Enter** button. A **↓** indicates that a new event cannot be entered while the **↓** is displayed.

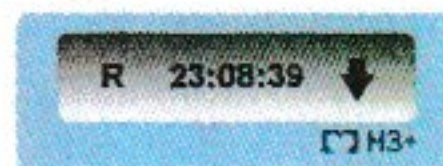

End of Recording

When the recording period has ended, the ID is displayed in reversed color indicating that the H3+ memory is full.



1. Remove the battery door.
2. Remove the battery and dispose of properly.
3. Replace the battery door.
4. Remove the patient cable from the recorder.
5. H3+ recorded data is ready to be transferred through connection of the USB cable.

Ending a Recording Early

The in-progress recording can be ended before the set duration is reached.

1. Remove the battery door.
2. Remove the battery.
3. Reinsert battery to display the ID in reversed color.
4. Replace the battery door.

H3+™ and Mortara® are trademarks or registered trademarks of Mortara Instrument, Inc.

2015-12



9503-165-02-ENG Rev H1

www.mortara.com ISO 13485 CERTIFIED

Mortara

Ref: 9503-165-02-ENG Rev H1